



# HEALTHCISER

## RESPIRATORY EXERCISER

SB Sales Healthciser is a Tri-balls incentive spirometer. The product is a breathing exerciser which is a flow-oriented device, provides an indirect indicator of the patient's inspired volume. It can be used to help strengthen respiratory musculature and to help restore and maintain lung capacity by encouraging slow, deep breathing.



### Features:

- Compact, high quality break-resistant plastic.
- Flow-oriented breathing trainer. The balls indirectly indicate patient's inhaled volume.
- Can be used to strengthen respiratory musculature and help restore or maintain lung capacity by encouraging a slow and deep breathing.
- Three color-coded balls in each chamber provide a visual incentive for patients.
- Air will enter each chamber and raise the balls depending on the flow of air inhaled per second.
- Spirometer lung capacity specifications: 600ml/sec, 900 ml/sec and 1,200ml/sec.
- The whole set includes EVA tubing, Mouth Piece, Tri-Ball Incentive Spirometer.
- The lung exerciser is provided with a unique handle for easy holding while exercising
- It comes with a tube holder to hold the flexible tube while not in use
- The exerciser can be dismantled into parts for cleaning and disinfection
- Single-patient use to minimize the risk of cross contamination.

*USE: Place the device at mouth level, exhale normally before inhaling., Inhale thoroughly until lifting the three balls (if, possible), Hold inspiration as long as possible., Remove the mouthpiece and exhale., Repeat the process*

